

CATHEREEPLUS™ Series




Made in Japan  **STERILE EO** Sterilized by E.O.G.

Urethane Gel Adhesive




Urethane gel adhesive is highly moisture permeable. This high permeability prevents skin maceration, and ensures stable adhesion for long time. The urethane gel adhesive reduces damage to skin when it is removed.

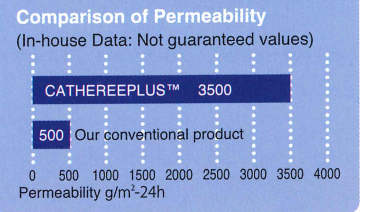

CATHEREEPLUS™




Usage Examples

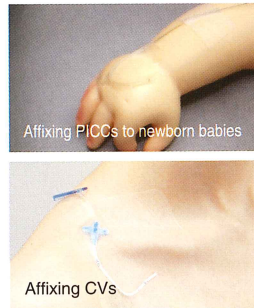



CATHEREEPLUS™ PAD




Usage Examples

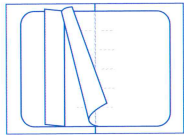

Product No.	Size	Per box
CPS 0203	25mm×30mm	30 sheets
CPS 0405	40mm×50mm	50 sheets
CPS 0608	60mm×80mm	35 sheets
CPS 0811	85mm×110mm	30 sheets
CPS 1114	110mm×140mm	25 sheets
CPS 1820	180mm×200mm	12 sheets

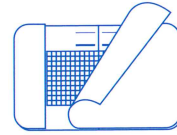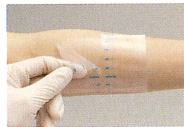

Product No.	Size		Per box
	Adhesive area	Pad	
CPSP 0508	50mm×80mm	25mm×50mm	50 sheets
CPSP 0610	60mm×100mm	30mm×65mm	50 sheets
CPSP 0812	80mm×120mm	40mm×80mm	25 sheets
CPSP 0915	90mm×150mm	50mm×100mm	20 sheets
CPSP 0921	90mm×210mm	50mm×160mm	15 sheets

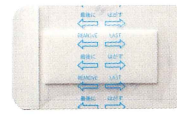


CATHEREEPLUS I.V.



Product No.	Size			Per box
	Adhesive area	Observation window	Catheter securing tape	
IV0608	60mm×80mm	34mm×39mm	15mm×80mm×3pcs	35 sheets
IV0811	80mm×110mm	44mm×51mm		30 sheets

CATHEREEPLUS I.V. SLIT

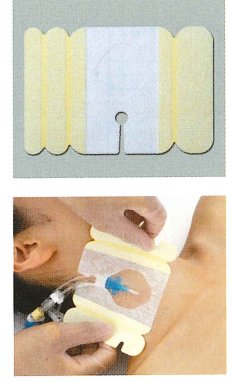

Product No.	Size			Per box
	Adhesive area	Observation window	Catheter securing tape	
IVS0709	70mm×90mm	34mm×39mm	12.5mm×90mm×2pcs 25mm×90mm×1pcs	50 sheets
IVS0810	80mm×95mm	44mm×51mm	23mm×47.5mm×4pcs	30 sheets

Highly Permeable Film Dressing (Permeability 3500g/m²-24h)

CATHEREEPLUS™ Series



Made in Japan




STERILE EO

Sterilized by E.O.G.

While Applied (Illustration)


CATHEREEPLUS (Urethane gel adhesive)

○: Water molecules

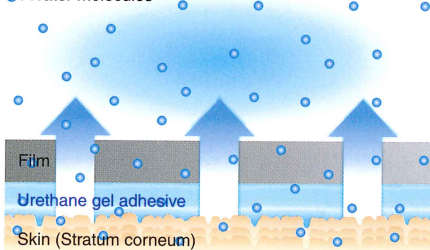

Because of the urethane gel adhesive with its high permeability, moisture such as sweat is able to escape easily while applied, making long-term adhesion possible.

Conventional product (Acrylic adhesive)

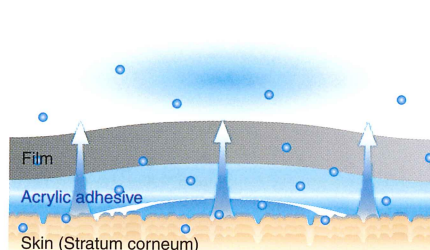


With acrylic adhesives, which has relatively low permeability, the tape easily comes off due to accumulation of sweat.

Highly Permeable Film Dressing

CATHEREEPLUS™ ROLL


Easy-to-Use Roll that is Suitable for Various Purposes

The slit for the release film has had a pre-lift process that makes it easy to remove.

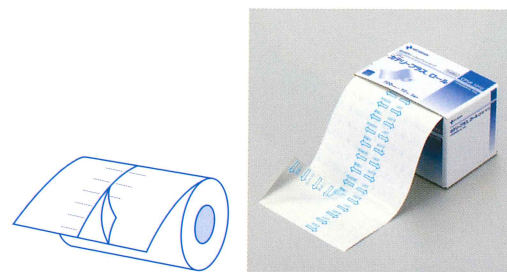


When Removing (Illustration)

CATHEREEPLUS (Urethane gel adhesive)

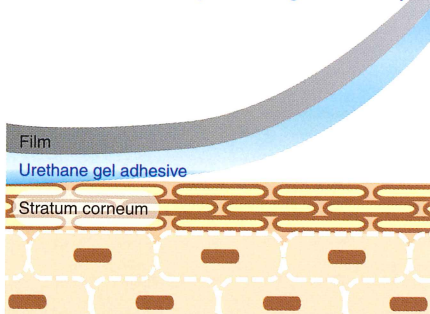


The adhesion and high permeability from the unique moist texture of the urethane gel adhesive prevents skin maceration, and reduces damage to skin when removing.

Conventional product (Acrylic adhesive)

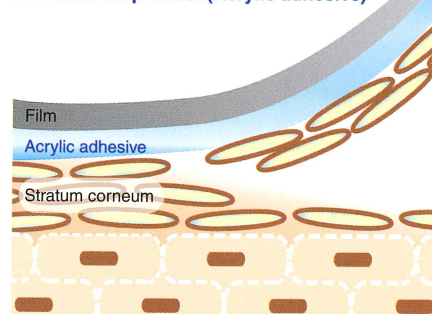

Because sweat is retained, the bonding of the stratum corneum is weakened. As a result, corneum peeling and epidermal peeling tend to occur when removing.

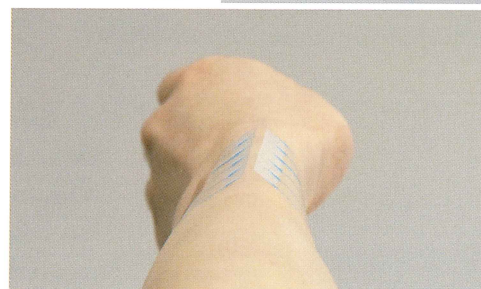


Product No.	Size	Quantity Per Package
CPSR 0510	50mm×10m	1
CPSR 1010	100mm×10m	1
CPSR 1510	150mm×10m	1

Hypoallergenic, Gentle on Skin

The combination of the skin-friendly adhesive and soft base material helps to reduce the peeling of skin cells and other damage, making it perfect for people with sensitive skin and for children who perspire.

Comparison Photos of Peeled Skin Cells Indigo colored specks are skin cells.

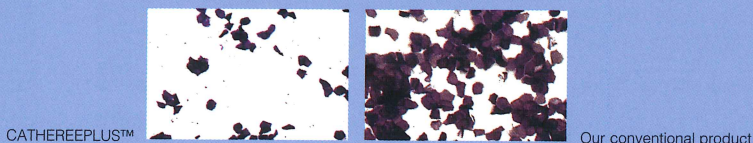

Contraindications / Prohibitions

Do not reuse.

Cautions for Usage

1. Important Basic Cautions

* While using this product, if a clinical infection occurs at the wound site, stop usage and administer proper treatment.

* While using this product, if symptoms appear that seem to indicate a skin problem (rash, redness, itchiness, etc.), stop usage and administer proper treatment.

* While using this product, if it becomes wet or dirty, or if there is bleeding or exudate, replace it with a new item immediately.

2. Other Cautions

* Do not use items from the package that is dirty, wet, or damaged.

* Use immediately after opening the package.

* When removing this product after using, remove it slowly in the direction of the body hair so that the skin is not injured.

Cautions for Storage

* Store at room temperature away from heat, humidity, direct sunlight, and water.

FOR BETTER BEING
いつも笑顔でいたいから——ニチバン

NICHIBAN

2-3-3 Sekiguchi, Bunkyo-Ku, Tokyo 112-8663

Homepage: <http://www.nichiban.co.jp/>

TM: trademark of NICHIBAN